



## Care Independence

Sling Model: .....

Serial Number: .....

User Name: .....

*The sling model and serial number  
are located on the label on the reverse of the sling.*

THIS USER GUIDE IS APPLICABLE  
FOR THE FOLLOWING  
GLOVE SLING® TYPES ONLY:

GB : GLOVE Bathing  
GBA : GLOVE Bathing with aperture  
GI : GLOVE In-Chair  
GIL : GLOVE In-Chair LITE

ISSUE DATE: 22/03/24  
VERSION: UG/GINCHHAM-EN04  
STATUS: APPROVED  
AUTHORISED BY: Technical Director

.....  J. ASH



Thank you for choosing a Care & Independence GLOVE SLING®.

We trust both you and the end-user will find the sling easy and comfortable to use throughout its lifetime.

In the event that you are dissatisfied with the sling performance or have any queries relating to the sling usage that are not addressed in this User Guide, please contact Care & Independence.

Care & Independence Ltd is a UK manufacturer registered with MHRA as a Class 1 Medical Device Manufacturer and this is a Class 1 medical device as defined in Article 51, Annex VIII Rule 1 of the Medical Device Regulation 2017/745.

These devices are designed and manufactured to meet or exceed BS EN 10535:2021 for the transfer of disabled persons and are designed and manufactured by Care & Independence, a certified ISO9001:2015 organisation.

#### Declaration

Care & Independence Ltd, declares under its sole responsibility that the aforementioned products to which this declaration relates, and which bear the CE marking, are in conformity with the applicable requirements of EU regulation 2017/745 as transposed in the laws of the Member States.

#### Intended Purpose

The device is intended to be used with hoisting equipment to allow a care giver, or the individual needing assistance, to reposition a person, to a different standing, or sitting position, to allow numerous activities to be completed.

# Glove Sling® USER GUIDE

## CONTENTS

1. General.....	Page 2
- Warranty	
- Compatibility	
2. Sling Safety.....	Page 2
- General	
- Sling Inspection	
3. Damaged or Faulty Slings.....	Page 2
4. Product Care.....	Page 2
- General	
- Laundering	
5. Getting to Know Your Sling.....	Page 3
- The In-Chair Hammock Style	
- Sling Labels Explained	
6. Fitting The Sling.....	Page 4
- From a Supine Position	
- Loop Strap -Leg Support Configuration	
7. Additional Support.....	Back cover

## 1. GENERAL



- 1.1 WARRANTY:** The GLOVE SLING® products covered in this User Guide have been awarded a LIFETIME WARRANTY. Any manufacturing defect will be rectified free of charge. Product damage through wear and tear, neglect or accidental damage is not covered by the warranty.
- 1.2 COMPATIBILITY:** Care & Independence slings have been manufactured to work with most of the major hoist manufacturers' equipment. A compatibility statement is available on request.

## 2. SLING SAFETY



- 2.1 GENERAL:**
- 2.1.1 Only competent persons should ever use hoists and slings.
- 2.1.2 A risk assessment must be completed before using the sling, even if the hoist and sling is provided from the same manufacturer.
- 2.1.3 It is essential that the sling is fit for purpose; the correct size and type of sling for an individual is essential to ensure the safety and comfort of the person being transferred.
- 2.1.4 Product care and inspection instructions should be followed at all times.
- 2.1.5 **No sling must ever be used to lift more than its rated load** (see label).
- 2.1.6 The intended sling and hoist combination should be compatible.
- 2.1.7 **If there is any doubt about sling safety, remove the sling from use until checked and authorised for use by a competent person.**

### 2.2 SLING INSPECTION:

- 2.2.1 Care & Independence recommend pre-use sling checks by competent person.
- 2.2.2 Slings are Class One Medical Devices and are subject to Lifting Operations and Lifting Equipment Regulations (LOLER) Examinations. A LOLER examination may only be conducted by a person trained and qualified to do so. Space is provided on the sling label to record LOLER details.



## 3. DAMAGED OR FAULTY SLINGS



Care & Independence have a strict *Returns & Repair Procedure* to ensure the safety of all involved.

**Please do not send slings and bathing products back without completing the Return/Repair form.**

## 4. PRODUCT CARE



### 4.1 GENERAL CARE:

- 4.1.1 Repeated washing may lead to sling deterioration. **Care & Independence strongly recommend replacing the sling at the first signs of wear or damage, as opposed to after a specified time frame.**
- 4.1.2 Each sling is marked with its maximum wash temperature.
- 4.1.3 Discolouration to some materials or fading labels may occur as a result of laundering. **NOTE: It is a health & safety requirement that the sling label must be legible throughout the sling's working lifetime.**

**4.2 WASHING THE SLING:** Every sling is marked with its maximum wash temperature. Close hook & loop fastenings before laundering to prevent damage to the fabric and webbing.

### 4.3 DECONTAMINATION:

Slings can withstand a strong decontamination routine but regular use may impair the lifespan of the sling.

**A copy of these instructions should be provided to the person(s) laundering the sling.**



- Wash gently at 40°C. Use a mild soap solution or a non-biological detergent without bleaching additives.
- Do not add bleach or stain removers.
- Air dry, cabinet dry or tumble dry on a cool cycle (below 60°C). Do not tumble dry on a hot setting.
- Do not place on hot pipes or heaters. Keep away from flames.
- Do not iron.


**FURTHER INFORMATION:** Available to download from our website



- Sling Care Instruction Guide (includes decontamination advice).
- Returns & Repair Procedure

## 5. GETTING TO KNOW YOUR SLING

### 5.1 THE IN-CHAIR HAMMOCK STYLE SLING

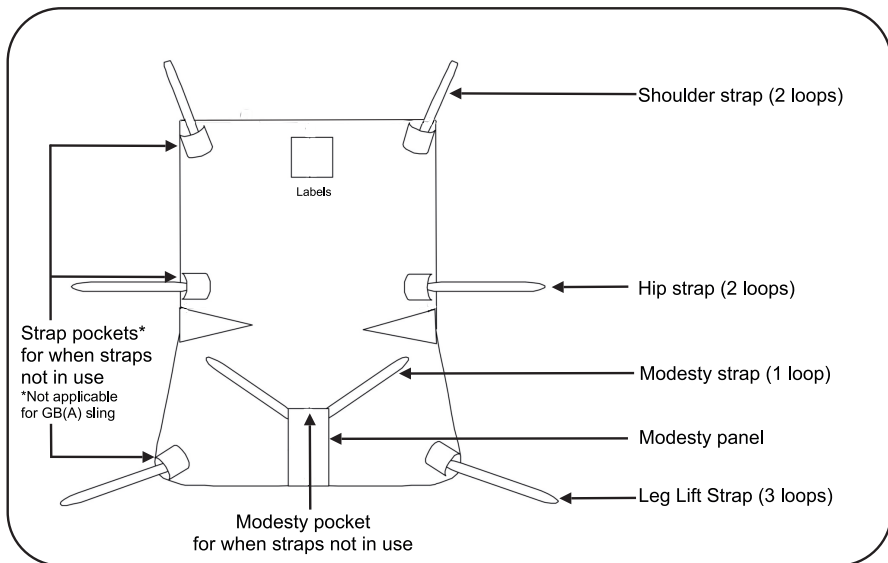

### 5.2 SLING LABELS EXPLAINED

There are three labels associated with each GLOVE SLING® product.


QR CODE: LEADS TO PRODUCT INFORMATION AND USER GUIDES	CARE & INDEPENDENCE CONTACT INFO
SLING INFORMATION YEAR OF MANUFACTURE MODEL SIZE PRODUCT CODE SERIAL NUMBER	LOLER RECORD INPUT
BAR CODES	
SAFETY INFO	
RECOMMENDED CARE INSTRUCTIONS	SWL IN KG
QUALITY MARKS & INFO	
NAME:	

#### 1 PRINTED LABEL

Printed label sewn onto back of sling. Provides useful information such as Sling Model Type, Washing instructions, Pre-Use Checks and a space to write User's name. A QR code is also featured that once scanned leads direct to product webpage where support resources are available. There is also space provided for a LOLER record.

#### 2 WOVEN LABEL

Woven label sewn onto back of sling. Contains serial number, Health & Safety Regulation Compliance mark, Maximum Load Capacity and QR code.

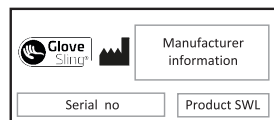

#### 3 QUALITY TAG (Disposable)

Attached to the side of the sling. Shows factory quality control approvals. Reverse side is Union flag.






## 6.1 FROM A SUPINE POSITION


**We do not recommend fitting this sling from a seated position.  
To prevent falls, always roll a person towards yourself, a wall or a bed rail.**

**NOTE:** This is an in-chair style sling. Subject to a risk assessment, it can be left under the seated user for an appropriate period of time.

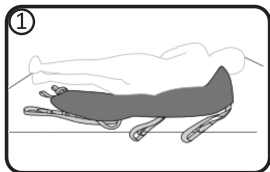


6.1.1

- i. Roll the user onto their side.
- ii. Fold sling in half lengthways with the label on the inside.
- iii. Position the sling with the top in line with the shoulders and the centre of the sling in alignment with the user's spine.

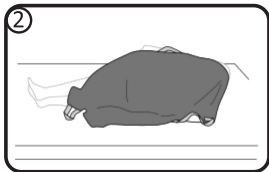


6.1.2

- i. Fold the uppermost section over the user like a blanket.
- ii. Tuck the remaining sling material under the user, ensuring straps are tucked in.

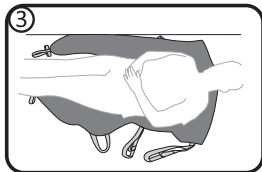


6.1.3

- i. Turn the user onto their opposite side.
- ii. Unfurl the sling beneath the user.
- iii. Reposition user onto their back with sling now spread out fully underneath them.

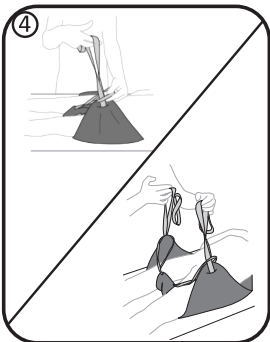

6.1.4


Feed the leg lift straps through the corresponding modesty straps.

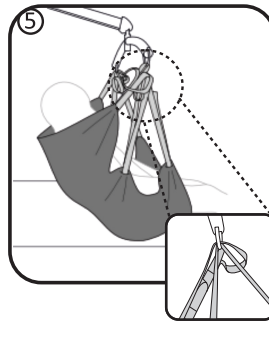

6.1.5

- i. Attach first leg lifts then hip straps and finally shoulder lift straps to the hoist spreader bar.
- ii. Perform a 'tug test'\*. After checking nothing is snagged, complete the transfer.

## 6.2 LOOP STRAP - LEG SUPPORT CONFIGURATION

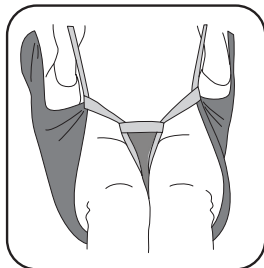

RECOMMENDED FOR GI, GIL & GB(A) SLING MODELS

Provides a comfortable and secure sitting position with the legs parallel.



It is important that the modesty panel is always used. **Never** attach the pommel straps to the spreader bar or hoist



\*A 'Tug Test' is recommended to ensure straps are properly secured to the spreader bar prior to hoisting. Manually pull on the straps in a downward motion.

# Glove Sling®

## USER GUIDE

Care & Independence  
www.careandindependence.com

ADDITIONAL SUPPORT AVAILABLE



SCAN SLING LABEL QR CODE FOR;

- Product information
- User Guide
- Fitting video
- Sling Care Instruction Guide
- Sling Pre-Use Inspection Guide
- Sling Sizing Guide



Free product training is also offered. Please contact us for further information.



- Expert advice
- Free assessments
- Full support



- Faded label replacement
- Sling repair
- LOLER examinations



Care & Independence Ltd,  
Unit 3, Telford Business Centre,  
Telford Road, Bicester, OX26 4LD

033 012 80808  
sales@careandindependence.com

*If a third party supplier has provided the GLOVE SLING®, you may like to write their contact details here:*



©2024 Care & Independence Ltd All Rights Reserved. Contents of this publication are intended as a guide only. Although due care and attention has been given to ensure that instructions are accurate and correct, in case of doubt, please refer to your prescribing clinician or contact the Care & Independence team on 033 012 80808. Care & Independence Ltd reserve the right to modify, add or delete details and is under no obligation to notify the user of amendment. All data is correct at time of publication.